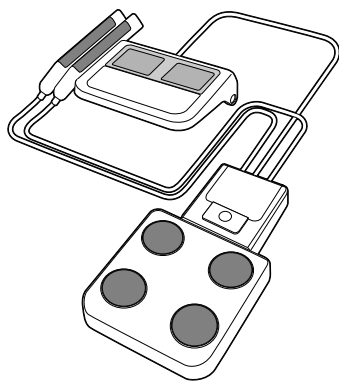
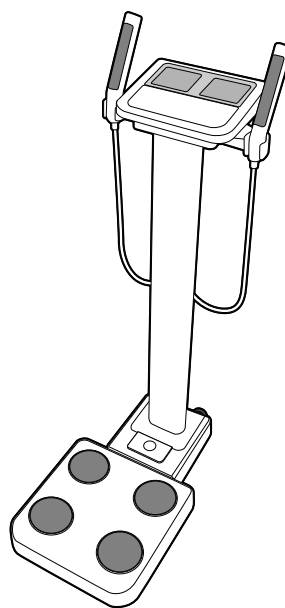
Description of Product Safety Information

BODY COMPOSITION ANALYZER MC-780MA-N

Separate Version

Column Mounted Version

Images in this manual are for illustration purposes only.

- 0122 This product meets the following requirements;
0123
1. Non-Automatic Weighing Instruments (2014/31/EU)
 2. Medical Device Directive (93/42/EEC)
 3. RoHS Directive (2011/65/EU)

<Manufacturer>

TANITA Corporation

1-14-2 Maeno-cho, Itabashi-ku,
Tokyo 174-8630 Japan
TEL: +81-(0)3-3968-7048
www.tanita.co.jp

<EU Representative>

TANITA Europe B.V.

Hoogoorddreef 56-E, 1101 BE
Amsterdam, the Netherlands
TEL: +31-(0)20-560-2970
FAX: +31-(0)20-560-2988
www.tanita.eu

TANITA Corporation of America, Inc.

2625 South Clearbrook Drive,
Arlington Heights, Illinois 60005
U.S.A.
TEL: +1-847-640-9241
FAX: +1-847-640-9261
www.tanita.com

TANITA Health Equipment H.K. Ltd.

Unit 301-303, Wing On Plaza, 3/F., 62
Mody Road,
Tsimshatsui East, Kowloon, Hong Kong
TEL: +852-2834-3917
FAX: +852-2838-8667
www.tanita.asia

TANITA (Shanghai) Trading Co., Ltd.

Room 8005, 877 Huai Hai Zhong Lu,
Shanghai,
The People's Republic of China
TEL: +86-21-6474-6803
FAX: +86-21-6474-7901
www.tanita.com.cn

<UK Representative>

TANITA Europe B.V.

111 Piccadilly, Manchester, M1 2HY, UK
TEL: +44-161-6380926
www.tanita.eu

Disposal

This equipment is an electronic device.
Please therefore dispose of it as an electronic
device instead of as general household
waste. Please follow regional regulations for
disposal.

Please read the Instruction Manual carefully and keep it for future reference.

Description of product

Intended Purpose

MC-780MA-N is a medical device to measure body composition, such as the percentage of body fat (fat percentage), using a noninvasive method on Bioelectrical Impedance Analysis (BIA), intended for the following use

- medical screening and health assessments
- monitoring the progress of weight loss during medical treatment

Indications

Relating to lifestyle diseases (diabetes, hyperlipidemia, bariatric surgery, hypertension and fatty liver disease).

Clinical Benefit

This product has been specifically designed to be simple to use and required no additional user assistance to take a measurement.

Measurements can be taken in under 30 seconds for maximum convenience.

Intended Patient Population

Healthy person (Excluding pacemaker and implantable cardioverter-defibrillator users).

Intended Users

Professional, no specialized facilities or expertise needed.

Contraindications

Pacemaker or implantable cardioverter-defibrillator users.

For your safety

This section explains precautionary measures to be taken to avoid injury to the patients and operators of this equipment, and to prevent damage to property. Please familiarise yourself with this information to ensure safe operation of this equipment.

The MC-780MA-N is not diagnostic equipment. In order to make an accurate diagnosis, in addition to the result of MC-780MA-N, the doctor in charge should conduct appropriate examinations and consider the results.

Contraindications

This equipment must not be used on subjects with pacemaker or implantable cardioverter-defibrillator users.

This equipment passes a weak electrical current through the body which could interfere with and cause the malfunction of electrical medical implants, resulting in serious harm.

Warning

Failure to follow instructions highlighted with this mark could result in death or severe injury.

Caution

Failure to follow instructions highlighted with this mark could result in injury or damage to property.

Prohibited

This mark indicates actions that are prohibited.

Required

This mark indicates instructions that must always be followed.

⚠ Warning**⊘ Prohibited****Do not handle the plug with wet hands.**

This may result in electric shock, fire, or current leakage.

**Do not modify this equipment.**

There is a risk of electrocution or injury, and precise analysis cannot be guaranteed.

**Do not use multiple adapters.**

This may result in fire.

**Do not use the equipment if you have any kind of wound or inflammation on any part of your body that comes into direct contact with the equipment.****⚠ Caution****⊘ Prohibited****Do not allow the equipment to get wet.****Avoid using on subjects with allergies to metals.**

Allergic reactions may be caused by the stainless steel used in the electrodes of this equipment.

Do not jump on the equipment.**Do not tilt the equipment.****Do not use this equipment near other products that emit electromagnetic waves.****Do not insert fingers into any of the gaps or holes.****Do not apply force to the display.**

The screen may break and cause injury.

Do not place items sensitive to magnetic forces near the equipment.

The magnet of the impedance meter may corrupt data on devices such as USB memory sticks if these are placed near the equipment.

Assist persons with disabilities.

Another person should assist persons with disabilities who may not be able to take a measurement alone.

⚠ Required**Clean the equipment after each use.**

Wipe off the equipment if dust accumulates or it becomes dirty.

Stand clear of the subject during measurement to ensure accuracy.**Continually monitor both the subject and the equipment for anomalies.**

If an anomaly in the subject or equipment is discovered, take appropriate action, such as stopping the equipment, while ensuring the safety of the subject.

Use the included AC adapter (TR30M120 or ATM036T-A120).**Do not lean against the equipment.****Unplug the AC adapter from the equipment when moving it.****Interpretation of analysis results (e.g. evaluation of measurements and formulation of exercise programs based on results) must be performed by a professional.**

Weight loss measures and exercise based on self-analysis could be harmful to your health.

Always follow the advice of a qualified professional.

This equipment is designated a Class B IT device (mainly for systems intended to be used in indoor environments) and is CE (EMC) certified, but it may affect devices that are sensitive to electromagnetic waves.

If connecting a computer or peripheral devices to this equipment, please use devices complying with IEC60601-1 (EN60601-1). Power must be supplied from a medical isolation transformer for IEC62368-1 (EN 62368-1) devices. Keep a distance of 1.5m between units during operation. Failure to do so may cause electric shock to subjects or malfunction.

If you do not intend to use this equipment for a long period of time, or if you always use AC adapter, remove the batteries before storage.**For accurate measurement****Note**

For people to whom any of the following apply, changes in measurement values should be referred to as a reference.

- People with metal implants
- People taking medications that induce changes in body water (e.g. diuretics, etc.)
- People who are pregnant, on dialysis or experiencing any swelling. The reliability of body fat percentages may decrease.
- Normal measurements cannot be performed for people with hard surfaces on their heel (stratum corneum). Moisten the sole before measuring.

⊘ Prohibited**Avoid measuring after strenuous exercise.**

This may cause inaccurate measurements. Please take measurements after sufficient rest.

**Avoid measuring after over-eating or over-drinking, or when severely dehydrated.**

This may cause inaccurate measurements. For greater accuracy, avoid using directly after waking up. Use at the same time of day each time, at least three hours after the last meal.

Ensure arms are not touching sides and inner thighs are not touching each other during measurement.**If necessary, place a dry towel between arm and side and/or between thighs.**

! Required

Use the equipment under the same conditions and in the same position as much as possible for accurate tracking of changes.

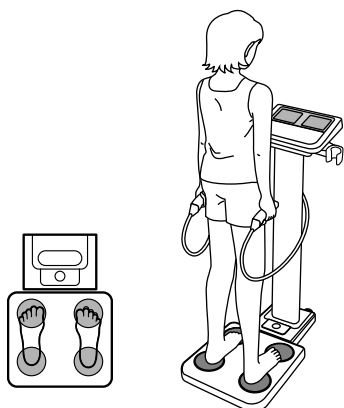
Readings are affected by the level of hydration and position of the body.

Please use at the same time of day each time, under the same conditions and in the same body position.

Avoid measuring in multiple locations with greatly differing temperatures.

This may cause inaccurate measurements. Allow the equipment to stand for at least 2 hours before using if it is moved to a new location with a temperature difference of 20°C/36°F or more.

Place palms, fingers and soles of feet in the correct positions on the electrodes when measuring.

Improper contact between the hands or feet and electrodes may cause the displayed fat percentage to be lower than the actual percentage, or cause an error.

Stand barefoot on the equipment after thoroughly washing hands and feet.

Proper measurements cannot be taken if wearing socks or if the palms or soles of the feet are dirty.

Do not sit or bend the elbows or knees.

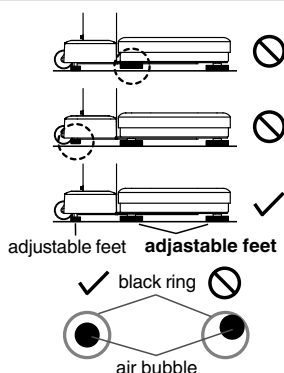
This causes inaccurate measurements.

Do not move during measurement.

This causes inaccurate measurements.

Use in a stable location.

Errors in measurements may occur if the equipment is used in an unstable location.

For further accuracy and security, please position the two adjustable feet on the base until they just make contact with the floor.

A portable RF communications equipment including antennas, can effect medical electrical equipment. It must not be used closer than 30 cm to any part of the ME equipment.

Use a cable of 3m or less when connecting each port to an external device.

Regular Maintenance

! Required

TANITA recommends that each facility conduct periodic checks of each unit.

- Check the following at least daily:
 - Check that the unit is on a stable and level surface, e.g. on firm flooring, not on a thick carpet
 - Date and time settings
- Visually inspect the following at least weekly:
 - Inspect the display for any damage or contamination
 - Inspect all cables, cords, and connector ends for damage or contamination
 - Inspect all safety-related labeling for legibility
 - Inspect all accessories (electrodes, etc.) for wear or damage
- Visually inspect the following at least monthly:
 - Mounting screws (optional)

Update settings, replace items, or call for service as necessary according to the results of the visual inspections. Do not use the unit if you see any signs of damage.

Equipment that has been damaged must be checked for proper operation by qualified personnel before using again.

Do not wipe the equipment with corrosive chemicals (benzene, cleaner, etc.).

Please use a mild detergent to clean the equipment.

Incorrect Ways to Measure

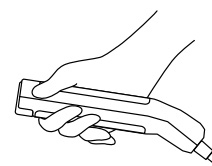
With bent knees

With bent elbows and wrists

With skin to skin contact

Without all fingers attached to the electrodes.

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